SONY

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English

▶ Getting started

See the illustration [a] ~ [d] in the Japanese text.

[a]-Earpieces

[b]-PR44P battery

[c]-Cleaning cloth

[d]-Carrying case

- 1 Open the battery compartment lid [A] and insert the battery (supplied).

Peel off the seal just before use.

- 2 Exchange the earpiece [E] if it does not fit your ear.

Use the accessory earpieces, which are larger or smaller than the original one, whichever fits your ear.

▶ Using the hearing aid

See illustration [1] in the Japanese text.

- 1 Slide the power switch [B] to “I” and turn on the power.
- 2 Put on the earphone [F].
- 3 Put the hanger [D] around your ear.
- 4 Adjust the volume with the volume button [C].

When you do not use the hearing aid

Slide the power switch to “O” and turn off the power.

If you cannot get enough hearing

Consult your dealer.

Battery life

The hearing aid operates for about 100 hours/TE-H30 or 80 hours/TE-H50 on one battery (Sony PR44P).

Adjusting to your ear

To change the position of the earpieces

See illustration [2] in the Japanese text.

The length of the hanger changes by 10 mm.

Notes

- Handle the hearing aid with care.
- Avoid moist or humid locations.
- After washing your hair, dry your hair sufficiently before you wear the hearing aid.
- Take off the hearing aid if you use hair spray.
- Do not leave the hearing aid in the kitchen or in the bathroom etc.
- Do not take a bath or have a sauna wearing the hearing aid.
- Do not leave the hearing aid in a location subject to direct sunlight, heat or moisture.
- Turn off the power when you do not use the hearing aid for a long time.
- Keep your ears and the earpieces clean.
- Remove stains with a dry cloth.
- If you use the hearing aid with or near a cellular phone, then some noise may occur.

If abnormal trouble occurs, turn off the power and consult your nearest Sony dealer.

Troubleshooting

Sound is not clear.

- ➡ Check battery polarity (+-).
- ➡ Replace the battery with new one.
- ➡ Turn up the volume.

Beeps occurs.

- ➡ Turn down the volume.

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Sony Corporation © 1996 Printed in Japan