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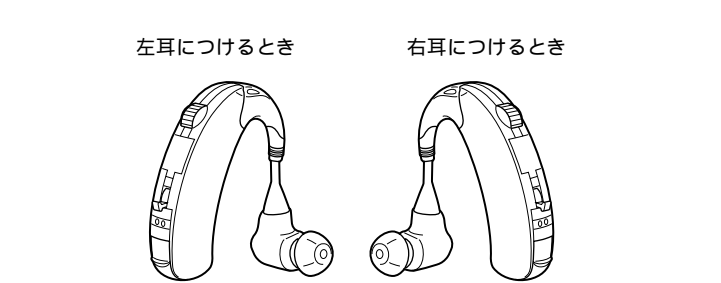
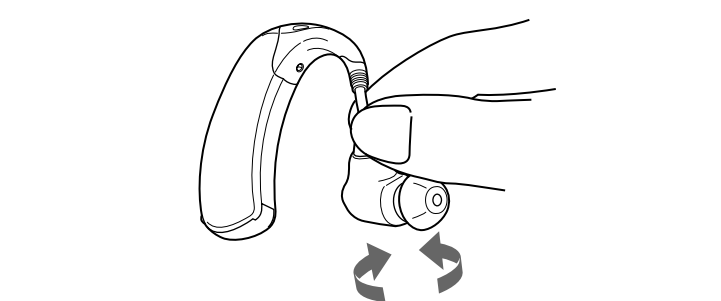
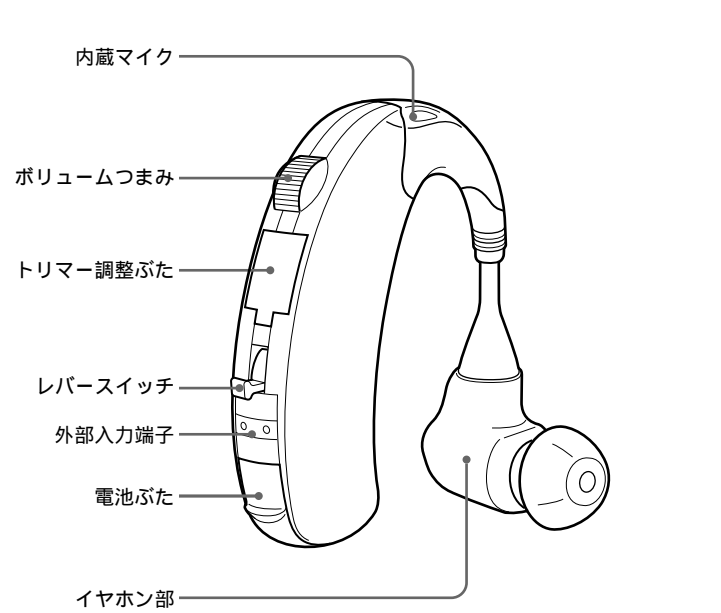
▶ Getting started

See illustration [\[1\]](#) in the Japanese text.

[A](#) Earpiece

[B](#) Audio adapter

[C](#) Sony PR44 (675) battery

[D](#) Cleaning cloth

[E](#) Carrying case

[F](#) Carrying pouch

[G](#) Spare filter

Fitting to your ear

Attaching the earpiece

If the size-M earpiece (attached at the factory) does not fit your ear, change it for one of S, L or LL sizes.

Inserting the battery

- 1 Open the battery compartment lid.
- 2 Remove the insulating sheet and insert the battery (supplied) as illustrated.
- 3 Close the battery compartment lid so that it clicks.

Battery life

The hearing aid operates for about 400 hours on the supplied Sony PR44 (675) battery.

▶ Using the hearing aid

See illustration [\[2\]](#) in the Japanese text.

- 1 Set the volume control to the “1” position.
- 2 Set the switch to the “M” position.
When you use the hearing aid in a noisy place, set it to the “S” (noise suppressing) position.
- 3 Put on the hearing aid with the earpiece fully inserted into your ear.
- 4 Adjust the volume to a moderate level.

After using
Set the switch to the “O” position.

Connecting to a TV, telephone, microphone, etc.

Using the supplied audio adapter, you can hear the sound from a TV, telephone, microphone, etc. via the hearing aid. Plug the hearing aid to the audio adapter as illustrated.

To connect to a TV or radio
Set the adapter's switch to the “LINE” position.

To connect to a microphone* or telephone
Set the adapter's switch to the “MIC” position.

* Sony ECM-A60 microphone designed for use with Sony's hearing aid is optionally available.