DVI-D Input Adaptor

取扱説明書 _____	JP
Instructions for Use _____	GB
Instructions d'utilisation _____	FR
Gebrauchsanweisung _____	DE
Istruzioni per l'uso _____	IT
Manual de instrucciones _____	ES
使用说明书 _____	CS
使用説明書 _____	CT
취급 설명서 _____	KR
Gebruiksaanwijzing _____	NL

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Before operating the unit, please read this manual thoroughly and retain it for future reference.

Important safeguards/notices for use in the medical environments

1. All the equipments connected to this unit shall be certified according to Standard IEC60601-1, IEC60950-1, IEC60065 or other IEC/ISO Standards applicable to the equipments.
2. Furthermore all configurations shall comply with the system standard IEC60601-1-1.
Everybody who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore, responsible that the system complies with the requirements of the system standard IEC60601-1-1.
If in doubt, consult the qualified service personnel.
3. The leakage current could increase when connected to other equipment.
4. For this particular equipment, all accessory equipment connected as noted above, must be connected to mains via an additional isolation transformer conforming with the construction requirements of IEC60601-1 and providing at least Basic Insulation.
5. This equipment generates, uses, and can radiate radio frequency energy. If it is not installed and used in accordance with the instruction manual, it may cause interference to other equipment. If this unit causes interference (which can be determined by unplugging the power cord from the unit), try these measures:
Relocate the unit with respect to the susceptible equipment. Plug this unit and the susceptible equipment into different branch circuit.

Consult your dealer. (According to standard EN60601-1-2 and CISPR11, Class B, Group 1)

WARNING

For the customers in the U.S.A.

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

You are cautioned that any changes or modifications not expressly approved in this manual could void your authority to operate this equipment.

All interface cables used to connect peripherals must be shielded in order to comply with the limits for a digital device pursuant to Subpart B of Part 15 of FCC Rules.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

For the customers in Canada

This Class A digital apparatus complies with Canadian ICES-003.

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Overview

The BKM-256DD DVI-D Input Adaptor is a DVI-D signal input adaptor for Sony professional monitors. When installed in an optional input slot on the main unit, it provides DVI-D input and output connectors for the main unit and a decoder for DVI-D signal.

Notes

- There are some monitors that are not compatible with this unit.
To check if the monitor is compatible with this unit, refer to the instructions for use of the monitor.
- DVI-A and DVI-I signals are not compatible with this unit.
- DVI-D dual-link signal is not compatible with this unit.

Function

Decoder for DVI-D signal

The BKM-256DD supports DVI-D single-link signal, which can be decoded by connecting a DVI-D 24-pin cable.

Cable equalizer

The BKM-256DD is equipped with an equalizer for received signals in the DVI-D signal receiver section.

5 V output function

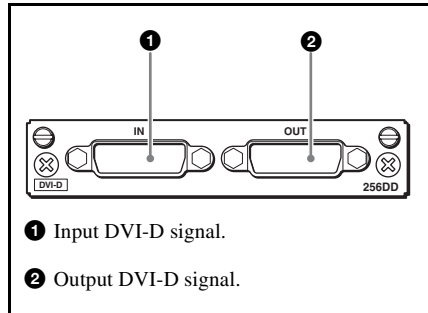
The BKM-256DD supplies 5 V power from pin 14 on either IN or OUT connector to an external device.

Using the Input and Output Connectors

For information about installing the BKM-256DD in an input option slot on the main unit, refer to the instructions for use of the monitor.

Configuration of input/output connectors and signals that may be input

The configuration of the input and output connectors, and the signals that may be input are shown below.

Input of DVI-D signal

You can input DVI-D signal to the connector **1**. Also you can output the signal, that input to the connector **1**, from connector **2**.

Note

The DC 5 V supply from the IN connector, and DC 5 V supply and output signal from the OUT connector are provided only when the power of the main unit is ON. For information about DC 5 V output, refer to the instructions for use of the monitor.

Specifications

General

Voltage +3.3 V, +5 V (supplied from the main unit)

Power consumption
Approx. 2 W

Operating conditions

Temperature
0 °C to 35 °C (32 °F to 95 °F)

Optimum temperature
20 °C to 30 °C (68 °F to 86 °F)

Humidity 0% to 90% (no condensation)

Pressure 700 hPa to 1060 hPa

Storage and transport conditions

Temperature -20 °C to +60 °C (-4 °F to +140 °F)

Humidity 0% to 90%

Pressure 700 hPa to 1060 hPa

Maximum external dimensions (w/h/d)
100 × 20 × 154 mm
(4 × ¹³/₁₆ × 6¹/₈ inches)

Mass Approx. 250 g (8.8 oz)

Input/Output connectors

DVI-D input/output
DVI-D 24-pin connector (2)

Available signal format

Refer to the instructions for use of the monitor.

Signal characteristics

DVI-D signal
Single-link TMDS compatible
Compliance with DVI Revision 1.0

Notes

- DVI-A and DVI-I signals are not compatible with this unit.
- DVI-D dual-link signal is not compatible with this unit.

Note

Depending on the SCAN setting of the monitor, part of the displayed picture may not appear.

Supplied accessories

Instructions for Use (1)
European Representative (1)

Design and specifications are subject to change without notice.

Note

Always verify that the unit is operating properly before use. SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.

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Sony Corporation