SONY REG 4-281-164-02 (2) 2017-04

HD/SD-SDI Input Adaptor

取扱説明書 <div>Instructions for Use</div> <div>Instruccions d'utilisacion</div> <div>Gebruiksaanwijsing</div> <div>Istruzioni per l'uso</div>	Manual de instrucciones <div>使用说明</div> <div>操作説明</div> <div>사용 설명</div> <div>Gebruiksaanwijzing</div>
--	---

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English
Before operating the unit, please read this manual thoroughly and retain it for future reference.
Important safeguards/notices for use in the medical environments
1. All the equipments connected to this unit shall be certified according to Standard IEC60601-1, IEC60950-1, IEC60065 or other IEC/ISO Standards applicable to the equipments.
2. Furthermore all configurations shall comply with the system standard IEC60601-1-1. Everybody who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore, responsible that the system complies with the requirements of the system standard IEC60601-1-1. If in doubt, consult the qualified service personnel.
3. The leakage current could increase when connected to other equipment.
4. For this particular equipment, all accessory equipment connected as noted above, must be connected to mains via an additional isolation transformer conforming with the construction requirements of IEC60601-1 and providing at least Basic Insulation.
5. This equipment generates, uses, and can radiate radio frequency energy. If it is not installed and used in accordance with the instruction manual, it may cause interference to other equipment. If this unit causes interference (which can be determined by unplugging the power cord from the unit), try these measures: Relocate the unit with respect to the susceptible equipment. Plug this unit and the susceptible equipment into different branch circuit.
Consult your dealer. (According to standard EN60601-1-2 and CISPR11, Class B, Group 1)
For the customers in the U.S.A.
This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

You are cautioned that any changes or modifications not expressly approved in this manual could void your authority to operate this equipment.

All interface cables used to connect peripherals must be shielded in order to comply with the limits for a digital device pursuant to Subpart B of Part 15 of FCC Rules.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

For the customers in Canada
This Class A digital apparatus complies with Canadian ICES-003.

For the customers in Europe
This product is intended for use in the following Electromagnetic Environments: E1 (residential), E2 (commercial and light industrial), E3 (urban outdoors), E4 (controlled EMC environment, ex. TV studio).

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Overview
The HD/SD-SDI Input Adaptor BKM-341HS is used for HD-SDI or SD-SDI signal input to Sony professional monitors. When the adaptor is installed in the option input connector on the rear of the monitor, it functions as an input connector that decodes HD-SDI or SD-SDI signals.
Note
Certain models of monitors are not supported by this adaptor. For information on compatibility with the adaptor, refer to the manual supplied with your monitor.

Configuration of Input/Output Connectors and Available Input Signals (Fig. A)

For information on installing this adaptor in the option input connector, refer to the manual supplied with your monitor.
① Digital input connector for inputting HD/SD-SDI signals (BNC type)
② Option output connector (D-sub 9-pin)

Specifications

General	
Power requirements	+5 V (supplied from the monitor)
Power consumption	Approx. 1.8 W
Operating conditions	
Temperature	0°C to 35°C (32°F to 95°F)
Optimum temperature	20°C to 30°C (68°F to 86°F)
Humidity	0% to 90% (no condensation)
Pressure	700 hPa to 1060 hPa
Storage and transport conditions	
Temperature	−20°C to +60°C (−4°F to +140°F)
Humidity	0% to 90%
Pressure	700 hPa to 1060 hPa
Maximum external dimensions (w/h/d)	Approx. 45 × 75 × 36 mm (1 7/8 × 3 × 1 1/2 inches) (not including the projected parts)
Mass	Approx. 120 g (4.2 oz)

Input/output connectors	
Digital input	BNC (1)
Option output	D-sub 9-pin (1)

Signal characteristics	
Input signal format	270 Mbps, 10 bit 4:2:2 component digital video
	• SMPTE 259M
	1.48 Gbps, 10 bit 4:2:2 component digital video
	• SMPTE 292M (except 1080/50p and 1080/60p input)

Accessories supplied
Instructions for Use (this document) (1)

Design and specifications are subject to change without notice.

Note
Always verify that the unit is operating properly before use. SONY WILL NOT BE LIABLE FOR DAMAGES OF ANY KIND INCLUDING, BUT NOT LIMITED TO, COMPENSATION OR REIMBURSEMENT ON ACCOUNT OF THE LOSS OF PRESENT OR PROSPECTIVE PROFITS DUE TO FAILURE OF THIS UNIT, EITHER DURING THE WARRANTY PERIOD OR AFTER EXPIRATION OF THE WARRANTY, OR FOR ANY OTHER REASON WHATSOEVER.

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© 2011 Sony Corporation